




HERIDA[®]
HEALTHCARE
 Moving & Handling Ltd



www.heridahealthcare.co.uk

HERIDA TRENT™ Classic Universal Sling

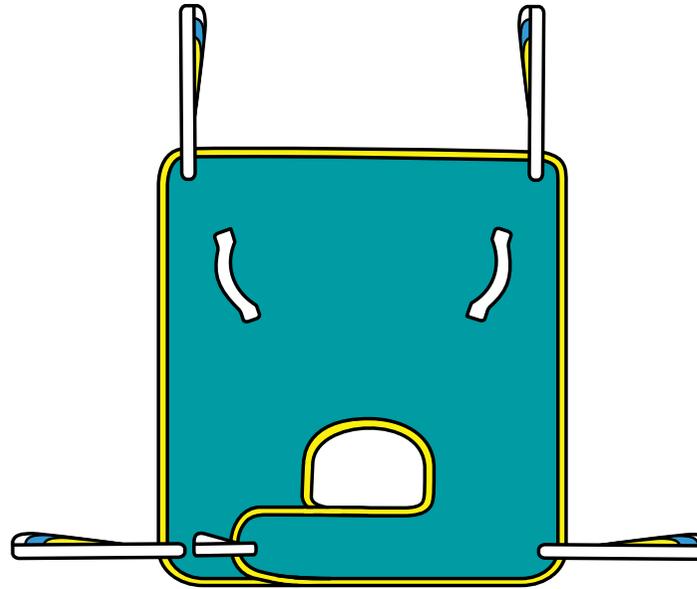

Flame retardant fabric used on all sling designs



Life time washable sling & label - LOLER compliant




Comprehensive warranty provided on all slings




BSEN10535 strength compliant


SIZE	WEIGHT LIMIT	PRODUCT CODE	BINDING	NOTES
Small	< 220kg / 35 stone	TRE*/POLY/S		Smaller / Larger sizes are available on request. *Insert relevant material to code NB: The Herida Trent™ sling selection is compatible for use with most mobile hoist and ceiling track products.
Medium	< 220kg / 35 stone	TRE*/POLY/M		
Large	< 220kg / 35 stone	TRE*/POLY/L		
X Large	< 220kg / 35 stone	TRE*/POLY/XL		
Special	< 220kg / 35 stone	TRE*/POLY/SPL		

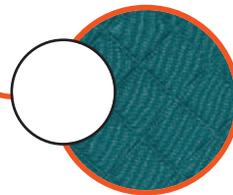


Quilted Teal Polyester

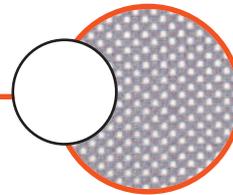


Grey Mesh Polyester

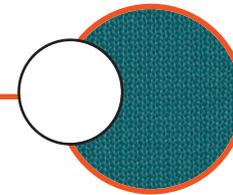

Teal Solid Polyester

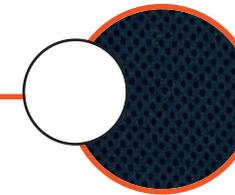

Black Spacer Fabric

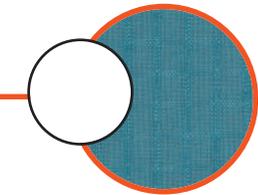

Blue Parasilk



HERIDA TRENT™

Classic Universal Sling

Do not attempt to lift any patient, during any circumstances without the appropriate training. Moving and handling vulnerable patients is of paramount importance and Herida does not endorse unsafe procedures or underqualified staff completing any lifting process. If you are unsure, please contact a Supervisor or Line Manager within your workplace, who is responsible for correct procedural lift and training policies on site.

CARE INSTRUCTIONS



Wash at temperature <90°C



Do not clean with bleach



Do not tumble dry



Do not iron



Do not dry clean

No fabric softener

No stain remover

Do not wipe with alcohol

Ensure Velcro® fastenings are fully attached before washing to avoid lint build up

Only use non-biological washing detergents and rinse thoroughly

Ensure back stays (if present) are always removed before washing



HERIDA
HEALTHCARE
Moving & Handling Ltd

HERIDA HEALTHCARE
MOVING & HANDLING LIMITED
Unit E1 & E2, Copley Hill Trading
Estate, Whitehall Road, Leeds,
LS12 1HE United Kingdom

Freephone: +44 (0) 800 193 6030

Email: sales@heridahealthcare.co.uk

Web: www.heridahealthcare.co.uk



Never use a loop fitting sling on a clip stud spreader bar and never use a clip fitting sling on a loop receiving spreader bar.



Please ensure that this User Instruction document and the accompanying "safe sling fitting and safety guide" are read prior to first use of any sling operation.

"A complete risk assessment must be completed by a competent and qualified professional!"

How to Fit a TRENT Classic Universal Sling from a sitting position



1. Ask the patient to lean forward, supported by the carer as necessary.



2. Place the sling behind the patient's back in a seated position as shown. Avoid pinching any skin areas or nipping areas of body mass.



3. As the sling has been applied behind the patient, ease the leg sections carefully under each limb.



4. Feed the straps through the dignity hoop section as shown. If the sling is to be used on an amputee patient, please ensure the leg stump is accommodated safely within the housed leg section and that the centre straps do not rub on the leg.



5. As the sling and patient are comfortable, reduce the height of the spreader bar avoiding the patients face and head at all times.



6. Place equal coloured straps to either side of the spreader bar, ensuring that the correct and comfortable position is achieved. A trained carer must always conduct this activity.



7. Tighten the straps by hoisting under tension gently until lift is ready. Communicate the process to your patient throughout the procedure.

How to Fit a TRENT Classic Universal Sling from a lying position – Bed or floor Always raise the bed to waist height to minimise stress on the carers back.



1. Roll patient safely, using appropriate and safe moving and handling methods. If the patient needs assistance to roll over, this should be done at the shoulder and hip areas.



2. Once in position, roll one half of the sling and tuck tightly under the patient as shown, taking care not to pinch the skin.



3. Roll the patient over to the other side of their hip and unroll the sling, so that it's fully open, interfacing the floor surface or bed.



4. Place the patient carefully on their back and gently pull out any creasing. The patient's hands should be placed by their stomach or by their side during this process.



5. If your patient is on the floor, lower the spreader bar and open the legs of the hoist around the patient's head, if your patient is in bed, the legs of the hoist should be placed underneath the bed.



6. Place the leg support sections underneath the legs and feed the straps through the dignity hoop before applying to the spreader bar. You can then apply the head section straps to the spreader bar, using matching loop or clip colours.



7. Raise the spreader bar gently, ensuring that no nipping of the skin or unnecessary pain is transferred to the patient. Regular communication and re-assurance should be provided.

● Always use a TRENT Sling with a head section if lifting from a bed or the floor

● For further help or advice please contact Customer Services on Freephone: +44 (0) 800 193 6030