

**HERIDA<sup>®</sup>**  
**HEALTHCARE**  
**Moving & Handling Ltd**



[www.heridahealthcare.co.uk](http://www.heridahealthcare.co.uk)

# HERIDA CLYDE™ Universal Handling Belt

The **Clyde™** Handling Belt helps trained carers perform supported manoeuvres and transfers safely without causing injury to themselves or the user.

This device should not be used on any hoisting device and is a manual assist product only. The **Clyde™** is suitable to meet multiple patient demographics.

This product shows supporting handles, allowing the carer to assist safe transfer as required to aid rehabilitation. An extension buckle is positioned on the front.




Flame retardant fabric used on all handling belt designs




Life time washable handling belt & label - LOLER compliant



Comprehensive warranty provided on all handling belts

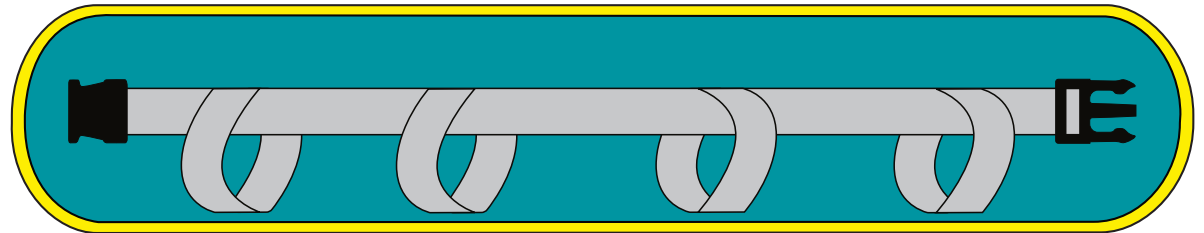

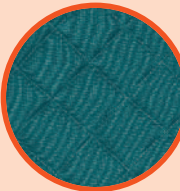SIZE	WEIGHT LIMIT	PRODUCT CODE	BINDING	NOTES
Small	< 150kg / 24 stone	CLY/POLY/S		<b>NB:</b> The Herida Clyde™ handling belt is not compatible for use on any hoisting device.   <b>Quilted Teal Polyester with a belt buckle as standard.</b>
Medium	< 150kg / 24 stone	CLY/POLY/M		
Large	< 150g / 24 stone	CLY/POLY/L		



# HERIDA CLYDE™

## Universal Handling Belt

### NB: Not for lifting

Do not attempt to move any patient, during any circumstances without the appropriate training. Moving and handling vulnerable patients is of paramount importance and Herida does not endorse unsafe procedures or underqualified staff completing any lifting process. If you are unsure, please contact a Supervisor or Line Manager within your workplace, who is responsible for correct procedural lift and training policies on site.

#### CARE INSTRUCTIONS



Wash at temperature <90°C



Do not clean with bleach



Do not tumble dry



Do not iron



Do not dry clean

No fabric softener

No stain remover

Do not wipe with alcohol

Ensure Velcro® fastenings are fully attached before washing to avoid lint build up

Only use non-biological washing detergents and rinse thoroughly



A full risk and safety assessment should always be conducted prior to any lift. Please ensure that this User Instruction document and the accompanying “safe sling fitting and safety guide” are read prior to first use of any sling operation. “A complete risk assessment must be completed by a competent and qualified professional”.

### How to Fit a CLYDE Universal Handling Belt



Never use a handling belt on any lifting device. Manually support the patient only, whilst considering your own health and posture, prior to that of those you are assisting.



1. Lean the patient forward slightly, allowing access to slip the stand aid/belt behind the back. As with all handling devices, support the individual as required. However, this item can only be used with patient groups that can maintain a high degree of support.



2. Maintaining a safe and dignified position as Carer, apply the belt, ensuring not to over tighten. Ensure that the hard buckle present on the design is clipped fully home and avoid pinching of patient skin.



3. Aid to the stand position by the carer, ensure that the patient is supported throughout.



4. The Patient should be supported from the side as shown or as per a qualified Carer. Communicate throughout to your patient.



**HERIDA®**  
**HEALTHCARE**  
Moving & Handling Ltd

HERIDA HEALTHCARE  
MOVING & HANDLING LIMITED  
Unit E1 & E2, Copley Hill Trading  
Estate, Whitehall Road, Leeds,  
LS12 1HE United Kingdom

Freephone: +44 (0) 800 193 6030

Email: [sales@heridahealthcare.co.uk](mailto:sales@heridahealthcare.co.uk)

Web: [www.heridahealthcare.co.uk](http://www.heridahealthcare.co.uk)

Issue 1 Jan 2019

● For further help or advice please contact Customer Services on Freephone: +44 (0) 800 193 6030